sonicBLU

BLUETOOTH HEADSET

Distributed by

Sonic Innovations, Inc.
2795 East Cottonwood
Parkway, Suite 660
Salt Lake City, Utah 84121
United States of America
(888) 678-HEAR (4327)

www.sonici.com

▶ PRODUCT USER GUIDE

www.sonici.com

Contact Details

NOiZFREE - earcare
Beranzerou 22
Athens 104 32 - Greece
Europe
tel: +30 210 5220.747
fax: +30 210 5235.601
www.noizfree.com
info@noizfree.com

Caution

This product is specifically designed for people with hearing difficulties and is provided with a special volume control for individual requirements. Proper care must be taken by all users that the volume control is set to the lowest level acceptable by each respective user. Care should therefore be exercised to ensure that any new user is familiar with this requirement.

Thank you

We would like to thank you for purchasing the sonicBLU Bluetooth® connectivity solution.

To fully appreciate the features of your new headset, please follow the steps in this User's Guide. The most appropriate person to help demonstrate this product to you is your hearing healthcare professional.

Introduction

sonicBLU is a wireless headset that allows you to make and receive calls while on the move or in the office. You can connect the sonicBLU headset to any compatible phone that supports Bluetooth® wireless technology.

Read this user guide carefully before using the sonicBLU headset. Read also the user guide for your phone, which provides important safety and maintenance information. Keep the headset out of reach of small children.

Bluetooth® wireless technology

Bluetooth® wireless technology makes it possible to connect compatible communication devices without using cables. A Bluetooth® connection does not require that the phone and the headset be in line of sight, but both devices should be within 10 meters (30 feet) of each other. Connections may be subject to interference from obstructions, such as walls or other electronic devices. There may be restrictions on using Bluetooth® technology in some locations. Check with your local authorities or service provider.

Battery information

Your device is powered by a rechargeable battery. The full performance of a new battery is achieved only after two or three complete charge and discharge cycles. The battery can be charged and discharged hundreds of times, but it will eventually wear out. Recharge your battery only with the included charger. Unplug the charger from the electrical plug and the device when not in use. Never use any charger that is damaged.

Leaving the device in hot or cold places, such as in a closed car in summer or winter conditions, will reduce the capacity and lifetime of the battery. Always try to keep the battery between 15°C and 25°C (59°F and 77°F). A device with a hot or cold battery may not work temporarily, even when the battery is fully charged. Battery performance is particularly limited in temperatures well below freezing.

Do not dispose of batteries in a fire as they may explode. Batteries may also explode if damaged. Dispose of batteries according to local regulations.

Language Index

English

English

Deutsch

Deutsch

Français

Français

sonicBLU

Bluetooth Headset User Guide

English

 Bluetooth®


Package Contents

1. One sonicBLU main unit
2. One sonicBLU Neckloop
3. One sonicBLU Dual Telecoil Hook Cable
4. Main Charger
5. MP3 Accessory/Extender Cable
6. User Guide

English

Basic Parts \ Main Unit

1. Charger Connector
2. Volume Control Key (increase)
3. Volume control key (decrease)
4. Multifunction Button/ Audio Plug
5. Indicator Light
6. Multifunction Key
7. Microphone

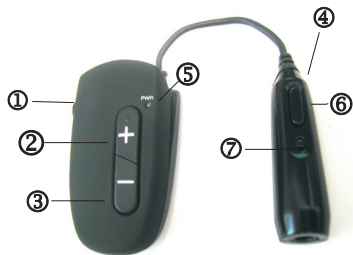How to Wear the Neckloop

Place the neckloop around your neck, then plug the connector into the Bluetooth® main unit.

Keep the main unit near your mouth so that the microphone on the main unit can pick up the sound of your voice.

How to Wear the Telecoil Hooks

Place the hooks on your ears next to your hearing instruments (see picture) then plug the connector into the Bluetooth® main unit.

Keep the main unit near your mouth so that the microphone on the main unit can pick up the sound of your voice.

Neckloop or Telecoil Hooks?

Your package contains both a neckloop and telecoil hooks. Experiment with both solutions and see which one offers you the best sound results and aesthetics for daily use.

How to Use the MP3 Accessory/Extender Cable

If you are using this cable as an extender cable, plug the accessory cable into the sonicBLU main unit and the Neckloop/Telecoil Hooks into the other end.

If you are using this cable for a sound source (e.g. an MP3 player), plug the cable directly into your audio player and the other end into your Neckloop/Telecoil Hooks. When used in this configuration the sonicBLU main unit is not used.

Charger & Charging

Before using the sonicBLU headset, you must first charge the battery.

Use only the charger found in the package. The use of any other charger may be dangerous.

Charge the battery - 1st charge for 2 hours!

This device has an internal, non-removable, rechargeable battery. Do not attempt to remove the battery from the main unit as you may damage the device.

1. Connect the charger to a wall outlet.

2. Connect the charger cable to the main unit.

The red indicator light is displayed during charging. If charging does not start, disconnect the charger and then plug it in again. Fully charging the battery may take up to 2 hours.

3. Completion of Charge

When the battery is fully charged, the red indicator light turns off.

Turning sonicBLU On or Off

To switch on:

Press and hold the multifunction key for 3 seconds. The blue indicator light will flash.

To switch off:

Press and hold the multifunction key for 3 seconds. The blue indicator flashes rapidly and then shuts off.

Pairing Instructions

Pairing sonicBLU with Your Bluetooth® Phone

You must pair the sonicBLU headset and your phone before you can make a call. "Pairing" creates a link between devices with Bluetooth® technology, and allows the phone to recognize the unique ID associated with your sonicBLU headset. When the sonicBLU headset and phone are paired, sonicBLU can automatically connect to your phone.

With the sonicBLU headset and phone in proximity:

1. With the headset off, hold the multifunction button until the indicator light continuously flashes blue-red (about 5 seconds).
2. Perform a device discovery from your phone (refer to the user guide that came with your phone). When the discovery is complete, the phone displays a list of found Bluetooth®-enabled devices.
3. Select "sonicBLU" from the list and follow any on-screen prompts.
4. When prompted, enter "0000" as the password and confirm. When the pairing is complete, the blue indicator flashes rapidly.
5. Your phone is now paired with the sonicBLU headset.

Disconnecting & Reconnecting

To disconnect the sonicBLU headset from your phone (for example, to connect your phone to another Bluetooth® device), do one of the following:

- Turn off the headset.
- Disconnect the headset in the Bluetooth® menu of your phone.
- Move the headset more than 10 meters (30 feet) away from the phone. You do not need to delete the pairing with the headset to disconnect it.

Reconnecting the paired headset to your phone

To connect the headset to your phone, turn on the headset and make the connection in the Bluetooth® menu of your phone. You can set your phone to connect with the headset automatically when the headset is turned on.

Basic Use - Accept/ End calls

To answer or to end a call, you must have the sonicBLU headset and your phone paired and within range (10 meters) of each other.

To answer a call:

sonicBLU will alert you with a ringing signal.
Press the multifunction button to accept the call.

To end a call:

Press the multifunction button to end the call.

Basic Use - Adjusting the volume

To increase the volume, press the top volume button repeatedly until you reach the desired volume level.

To decrease the volume, press the bottom volume button repeatedly until you reach the desired volume level.

Advanced Use - Voice-Activated Dialing

Note: This feature is only available on phones that support the voice-activated dialing feature.

To perform a voice-activated dial:

1. Quickly press the multifunction button 2 times.
2. Speak the recorded name of your contact. The phone will dial the phone number associated with the spoken contact name.

English

Advanced Use – Reject a Call

To reject a call, press and hold the multifunction button when the phone rings.

Advanced Use – Answer Second Incoming Call

To accept a second incoming call, press and hold the multifunction button when the phone rings with the second call. This places the first call on hold and accepts the second call.

English

Troubleshooting

If you try to make a call but you cannot hear anything through your hearing instrument, make sure that:

- The sonicBLU headset is charged and turned on (the indicator flashes periodically).
- The sonicBLU headset and your phone are paired.
- The sonicBLU headset and your phone are connected.
- If using the telecoil hooks, make sure they are positioned correctly on your ear.
- Your hearing instrument is switched to a telecoil program.
- The volume is turned up.

English

Basic Use - Headset Indicators

Headset Indicator	Status
No indicator	Power off
Flashes blue-red	In pairing mode
Flashes rapidly	Pairing successful
Flashes every 3 seconds	Standby mode (not in call)
Flashes every 8 seconds	Connected

English

Basic Use - Audio Tones Indicators

Audio Tone	Headset Status
Series of 2 tones when pressing volume button	Volume reached maximum or minimum level

English

Basic Use - Audio Tones Indicators

Audio Tone	Headset Status
5 rapid high tones, repeated every 20 secs	Low battery
No audio indicators, deteriorating audio quality	Out of range
Ring tone	Incoming call
Single high pitch tone	Pairing confirmation
Single low pitch tone	Initiate voice activation
Single high pitch tone	End of call

English

Declaration of Conformity

NOiZFREE declares under its sole responsibility that the product sonicBLU is in conformity with the provisions of the following Council Directive: 1999/5/EC. A copy of the Declaration of Conformity can be found at their website www.noizfree.com

The crossed-out wheeled bin means that within the European union the product must be taken to separate collection at the product end-of-life. Do not dispose of these products as unsorted municipal waste.

Bluetooth® is a registered trademark of Bluetooth SIG, Inc

NOiZFREE operates a policy of continuous development. NOiZFREE reserves the right to make changes and improvements to any of the products described in this document without prior notice. Under no circumstances shall NOiZFREE be responsible for any loss of data or income or any special, incidental, consequential or indirect damages howsoever caused. The contents of this document are provided "as is". Except as required by applicable law, no warranties of any kind, either express or implied, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose, are made in relation to the accuracy, reliability or contents of this document. NOiZFREE reserves the right to revise this document or withdraw it at any time without prior notice.

Declaration of Conformity

FCC Notice to Users

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will occur in a particular installation. If the equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and the receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Declarations

Electromagnetic Interference/Compatibility

Note: Nearly every electronic device is susceptible to electromagnetic interference (EMI) if inadequately shielded, designed, or otherwise configured for electromagnetic compatibility.

Facilities

To avoid electromagnetic interference and/or compatibility conflicts, turn off your headset in any facility where posted notices instruct you to do so. Hospitals or Health care facilities may be using equipment that is sensitive to external RF energy.

Aircraft

When instructed to do so, turn off your headset when on board an aircraft. Any use of the headset must be in accordance with applicable regulations per airline crew instructions.

Medical Devices

Pacemakers

The Health Industry manufacturers Association recommends that a minimum separation of 15 centimeters (6inches) should be maintained between a handheld wireless device and a pacemaker. These recommendations are consistent with the

independent research by, and recommendations of, Wireless Technology Research.

Persons with pacemakers should:

- ALWAYS keep the phone and Bluetooth® headset more than 15 centimeters (6inches) from your pacemaker when the phone is turned ON.
- NOT carry the phone and Bluetooth® headset in the breast pocket. Turn OFF the phone and headset immediately if you have any reason to suspect that interference is taking place.

Hearing Aids

Some digital wireless phones may interfere with some hearing aids. In the event of such interference, you may want to consult with your hearing healthcare professional or your hearing aid manufacturer to discuss alternatives.

Other Medical Devices

If you use any other personal medical device, consult the manufacturer of your device to determine if it is adequately shielded from RF energy. Your physician may be able to assist you in obtaining this information.

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Bluetooth® est une marque déposée de Bluetooth SIG, Inc